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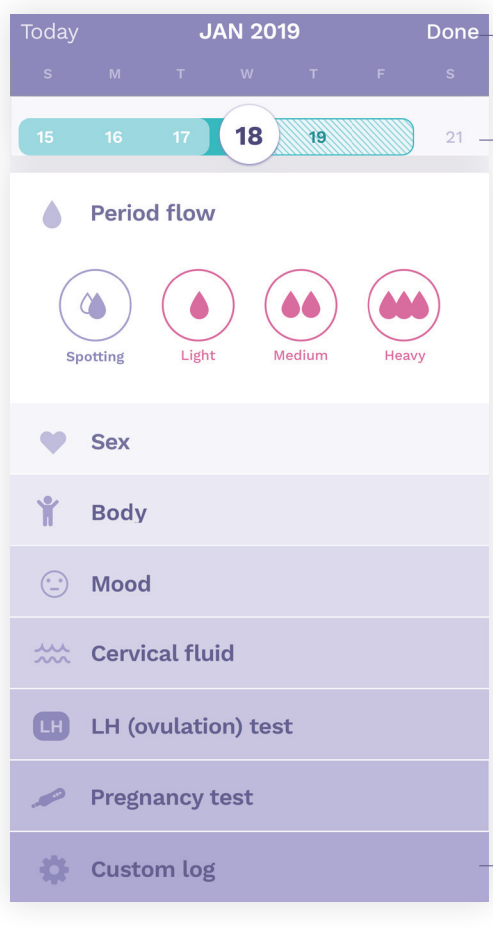
The screenshot shows the Ava app interface on an iOS device. At the top, there's a calendar header for 'JAN 2019' with days of the week (S, M, T, W, T, F, S) and dates (15, 16, 17, 18, 19, 20, 21). The date 18 is highlighted. Below the calendar, there's a 'Period flow' section with four icons: Spotting, Light, Medium, and Heavy. Below that, there are sections for Sex, Body, Mood, Cervical fluid, LH (ovulation) test, Pregnancy test, and Custom log. A hand icon points to the date 18, indicating a selection action. A line points from the 'Period flow' section to the text 'Dodaj miesiączkę, stukając w symbol. Stuknij ponownie, aby usunąć.' Another line points from the 'Custom log' section to the text 'Dodatkowe opcje zapisu'.

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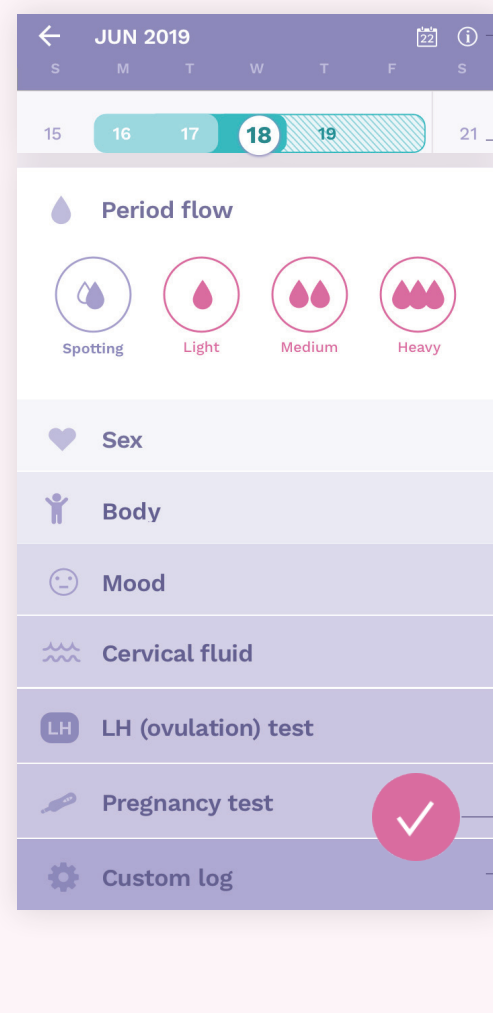
The screenshot shows the Ava app interface on an Android device. At the top, there's a calendar header for 'JUN 2019' with days of the week (S, M, T, W, T, F, S) and dates (15, 16, 17, 18, 19, 20, 21). The date 18 is highlighted. Below the calendar, there's a 'Period flow' section with four icons: Spotting, Light, Medium, and Heavy. Below that, there are sections for Sex, Body, Mood, Cervical fluid, LH (ovulation) test, Pregnancy test, and Custom log. A hand icon points to the date 18, indicating a selection action. A line points from the 'Period flow' section to the text 'Dodaj miesiączkę, stukając w symbol. Stuknij ponownie, aby usunąć.' Another line points from the 'Custom log' section to the text 'Dodatkowe opcje zapisu'. A line points from the 'Pregnancy test' section to the text 'Potwierdź zapis'. A line points from the 'Custom log' section to the text 'Stuknij, aby utworzyć własny niestandardowy dziennik. Możesz utworzyć maksymalnie 10 dzienników'.

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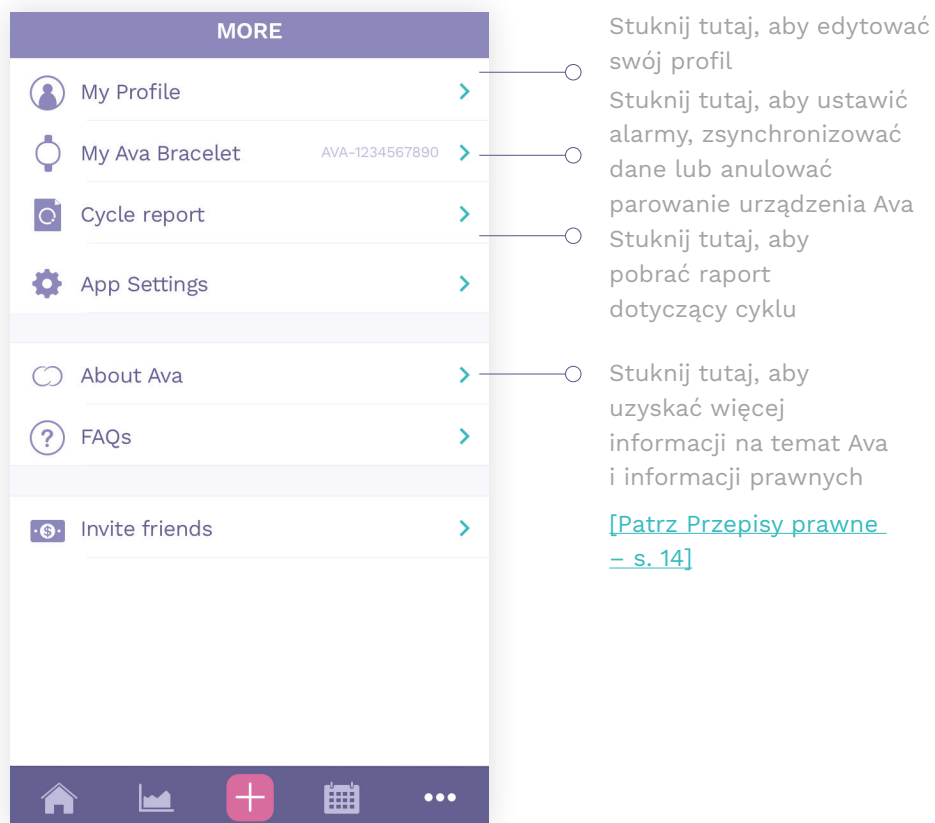
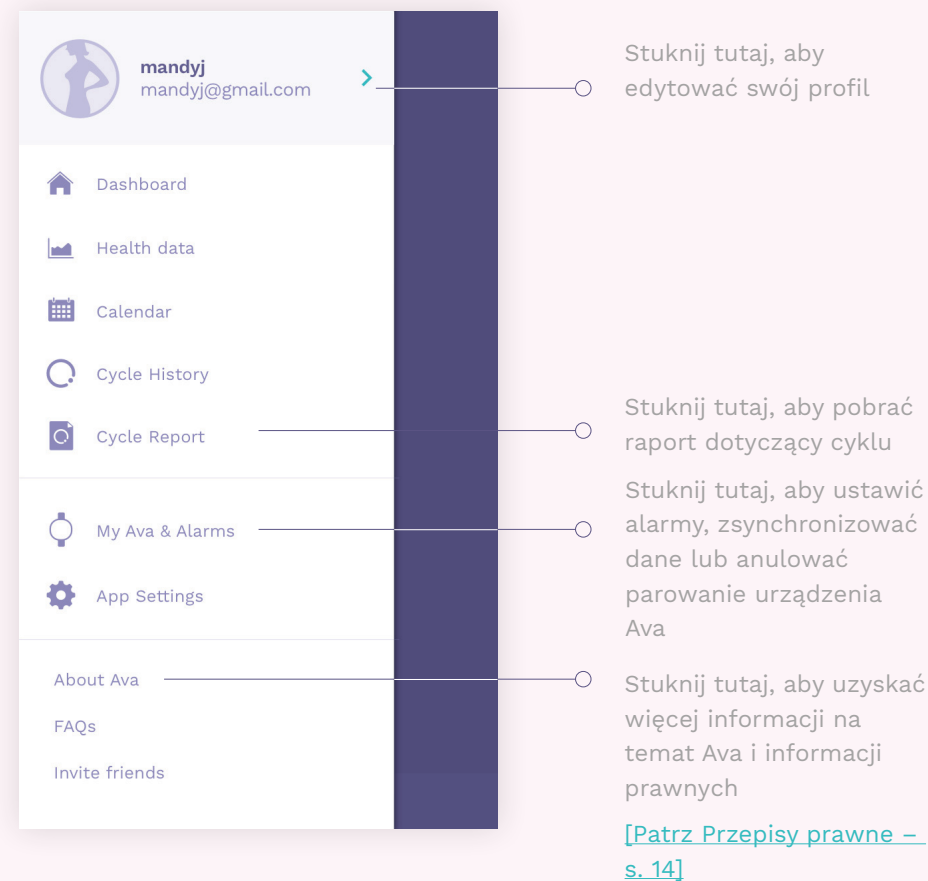
mandyj
mandyj@gmail.com

MY CURRENT STATUS

☐ I'm tracking my cycle ☒ I'm trying to conceive ☐ I'm pregnant

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< Profile CYCLE REPORT

Download your personal cycle report

The cycle report provides an overview of your last 12 cycles in a printable PDF format. Save it for yourself or share it with your doctor.

Only cycles in which you've synced data will be included in this report.

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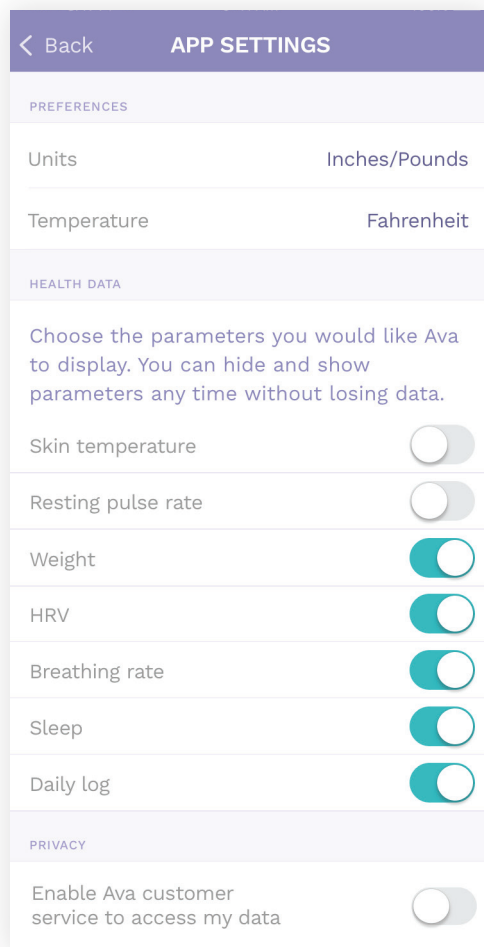
Temperature Fahrenheit

HEALTH DATA

Choose the parameters you would like Ava to display. You can hide and show parameters any time without losing data.

Skin temperature ☐

Resting pulse rate ☐

Weight ☒

HRV ☒

Breathing rate ☒

Sleep ☒

Daily log ☒

PRIVACY

Enable Ava customer service to access my data ☐

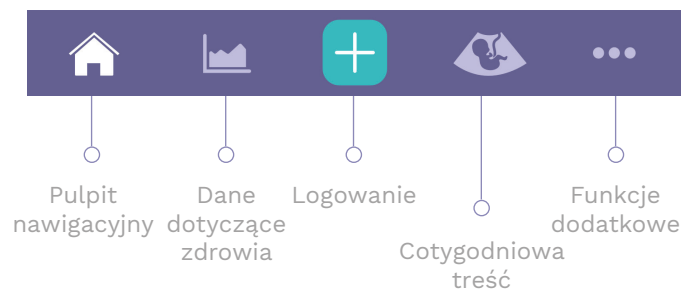
A high-angle, top-down photograph of a pregnant woman with dark curly hair, wearing a grey long-sleeved cardigan, sleeping peacefully in a bed. She is lying on her side, with her head resting on a white pillow and her hand near her face. Her left wrist is resting on her belly, and a small, light blue, circular sensor is attached to her wrist. The bed is covered with a grey duvet and several pillows, including one with blue and white stripes. To the left of the bed, a wooden bedside table holds a white alarm clock and a stack of three grey cylindrical containers. The room is dimly lit, with soft light coming from the right side, creating a calm and intimate atmosphere.

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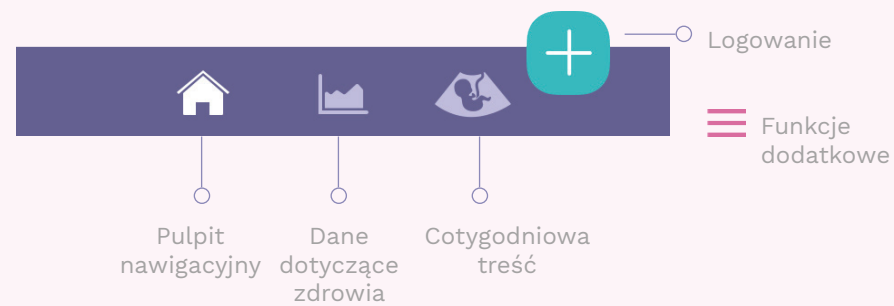
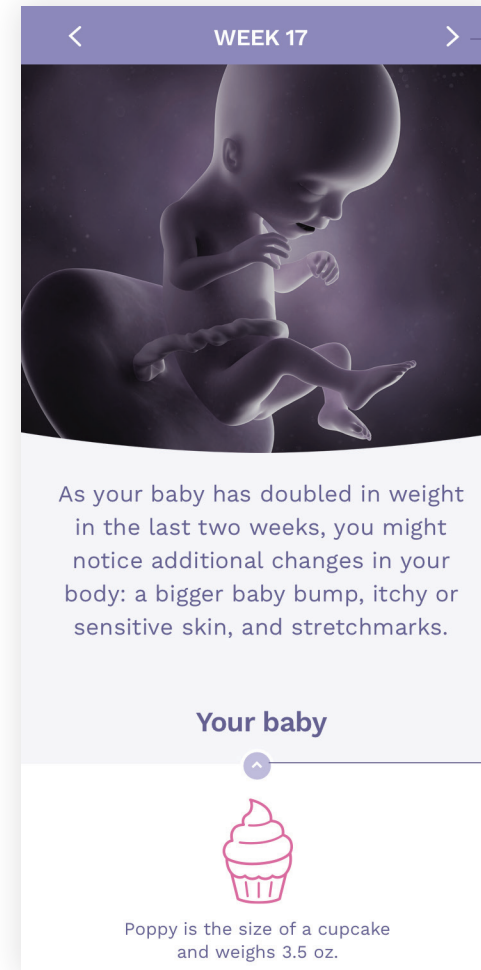
WEEK 17

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As your baby has doubled in weight in the last two weeks, you might notice additional changes in your body: a bigger baby bump, itchy or sensitive skin, and stretchmarks.

Your baby

Poppy is the size of a cupcake and weighs 3.5 oz.

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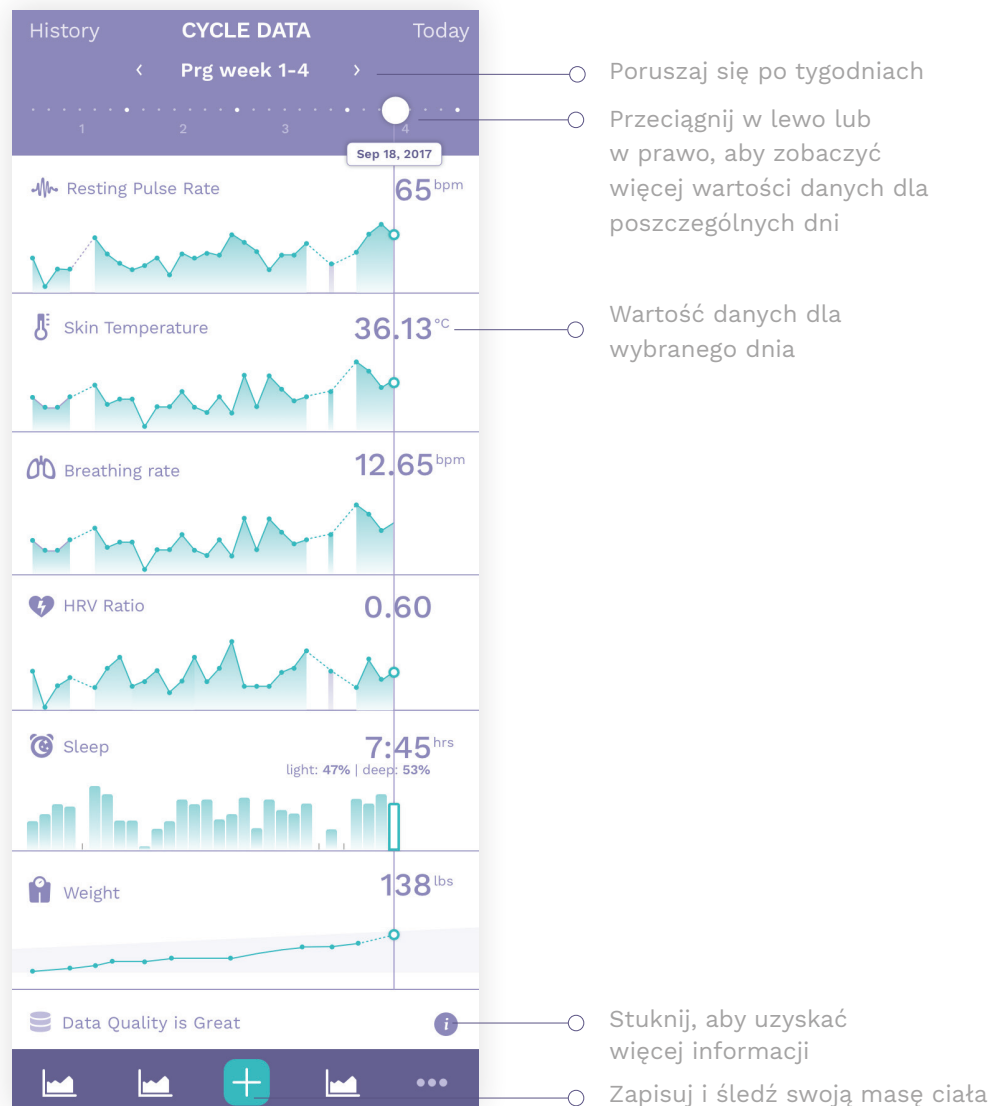
PROFIL

The screenshot displays the 'MY PROFILE' screen with the following sections:

- MY CURRENT STATUS**: Three options: 'I'm tracking my cycle', 'I'm trying to conceive', and 'I'm pregnant' (selected).
- MY PREGNANCY**: Due date (May 20, 2017) and Baby nickname (Poppy).
- ACCOUNT DETAILS**: First name (Amanda), Last name (Jones), and Password (change).
- Buttons: Log out, Delete account.

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© Sierpień 2021 AVA AG, wszelkie prawa zastrzeżone. Ava i logo Ava są znakami towarowymi firmy Ava AG. Bluetooth jest znakiem towarowym firmy Bluetooth SIG, Inc. iPhone i iOS są znakami towarowymi firmy Apple Inc. zarejestrowanymi w Stanach Zjednoczonych i innych krajach. Android jest znakiem towarowym firmy Google Inc. Wszystkie znaki towarowe niewymienione z nazwy są własnością ich odpowiednich właścicieli.