

Tracy Y Zhu¹, Brianna M Goodale^{2*}, Maureen Cronin², Martina Rothenbühler², Brigitte Leeners³

¹Department of Reproductive Endocrinology, University Hospital Zurich, Zurich, Switzerland; ²Ava AG, Zurich, Switzerland; ³University Hospital Zurich, Zurich, Switzerland
*Presenting author

Introduction

- More than 60% of women do not know when they can conceive during their menstrual cycle.¹ Accurately identifying the fertile window and properly timing intercourse can increase the probability for conception.
- Urine-based Luteinizing Hormone (LH) kits can routinely be used to detect the rise in LH occurring 24-36 hours prior to ovulation, identifying only the end of the fertile window.²
- The Ava Fertility Tracker is a wearable device measuring five physiological parameters to identify an average of 5 days in the fertile window prospectively with 90% accuracy.³
- **Research aim:** determine the accuracy of the Ava Fertility Tracker compared to urinary LH tests.

Methods

- This study included 205 ovulatory cycles, confirmed by LH tests, from 61 women who wore the Ava Fertility Tracker nightly for up to six cycles.
- The Ava Fertility Tracker measured pulse rate, respiratory rate, skin perfusion, heart rate variability, and skin temperature during sleep and employed two machine learning algorithms to track fertility in a smartphone application:
 1. **Retrospective algorithm** determining the fertile window after cycle end
 2. **Prospective algorithm** predicting the fertile window in real time
- Ovulation day was defined as the day following a positive LH test; the fertile window included ovulation day and the five preceding days.
- Accuracy was determined based on the number of wrongly identified ovulation days as compared to LH test results tolerating ± 2 days (i.e., equivalence interval).
- Sensitivity and specificity of the algorithms were calculated by means of a mixed effects model.



Figure 1. Ava Fertility Tracker and its mobile application.

References


1. Johnson SR, Pion C. Multinational survey of women's knowledge and attitudes towards fertility and pregnancy. RCOG World Conference Liverpool, UK; 2013. p. 194.
2. Singh M, Saxena BB, Rathnam P. Clinical validation of enzyme immunoassay of human luteinizing hormone (hLH) in the detection of the preovulatory luteinizing hormone (LH) surge in urine. Fertil. Steril. 1984;41:210–217.
3. Goodale BM, Shilahi M, Falco L, Dammeier F, Hamvas G, Leeners B. Wearable sensors reveal menses-driven changes in physiology and enable prediction of the fertile window: Observational study. J Med Internet Res. 2019;21(4):e13404. doi:10.2196/13404.
4. Wilcox AJ, Weinberg CR, Baird DD. Timing of sexual intercourse in relation to ovulation: Effects on the probability of conception, survival of the pregnancy, and sex of the baby. NEJM. 1995; 333(23): 1517–1521. doi:10.1056/NEJM199512073332301

Results

- **Mean participants' age:** 26.5 \pm 4.2 years
- **Mean duration of included cycles:** 29.9 \pm 3.2 days

Table 1. Accuracy of the two algorithms using LH test results as reference.

Variables	Retrospective Algorithm	Prospective Algorithm
Accuracy to identify ovulation day, no. (%)		
Cycles with errors ± 2 days	168 (82)	148 (72.2)
Cycles with 0 days errors	57 (27.8)	126 days (87)
Cycles with errors larger than ± 2 days	37 (18)	57 (27.8)
Accuracy to label fertile/infertile days (95% confidence interval)		
Sensitivity	0.80 (0.76, 0.83)	0.77 (0.71, 0.82)
Specificity	0.95 (0.94, 0.95)	0.91 (0.90, 0.92)

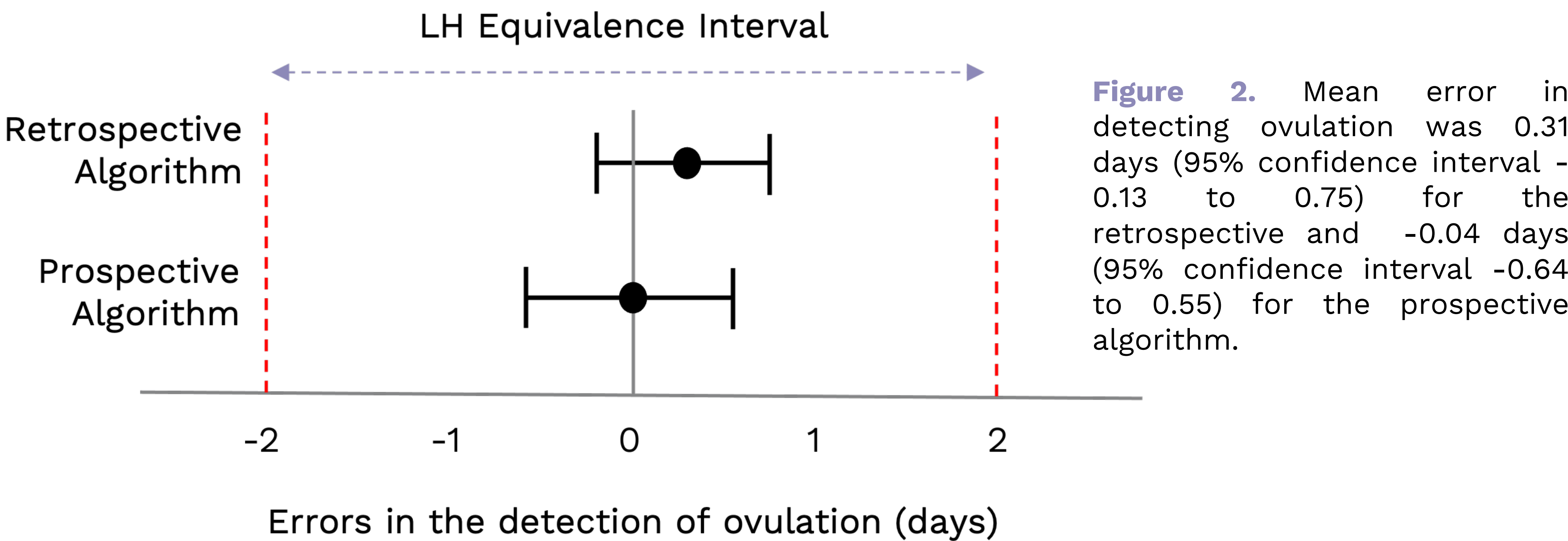

Conclusion

- The Ava Fertility Tracker was as accurate in identifying ovulation day as LH tests, suggesting it could serve as a reliable marker of ovulation for real-world users as well as researchers.
- Leveraging machine learning and wearable sensor technology, the Ava Fertility Tracker also provides a larger and prospective fertile window than LH tests thereby enabling women to increase their chances of conceptive sex.⁴