AVA FERTILITY TRACKER

User Manual

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EN – DIRECTIONS & SAFETY INFORMATION

! HEALTH WARNINGS

- This product is not for contraceptive use.
- Do not use if you suffer from cholinergic urticaria (sweat induced allergies).
- This device is not to be used for fever measurement and to diagnose, treat, cure, or prevent any disease.

! ADVERSE REACTIONS - WARNINGS

- Wearing the device snugly for extended time can lead to skin irritation or sensitization due to rubbing and friction. If signs of an allergic reaction occur or local irritation persists, please discontinue use and contact your physician.
- If you feel soreness, tingling, numbness, burning or stiffness in your hands or wrists while or after wearing the product, please discontinue use and contact your physician.

! HEALTH / MEDICAL PRECAUTIONS

- Remove your device if it feels hot.
- Only limited data are available for cycles that are shorter than 24 or longer than 35 days, which might result in incomplete or inaccurate predictions in users with shorter or longer

cycle lengths.

- Only limited data are available for use in case of diagnosed or clinical infertility, which might result in incomplete or inaccurate predictions in users with these conditions.
- The following circumstances may affect you and your body and could result in inaccurate Ava readings: Irregular cycles / Irregular sleep / Working night shifts / Insomnia / Use of temperature-affecting medications / Alcohol / Feeling hungover / Extensive smoking / Recreational drug use / Feeling sick / Infections / Extreme physical or mental stress / Travelling through different time zones.
- Consult your physician if you are not yet pregnant after six months of actively trying to conceive using this device.
- Consult your physician prior to use if you have tendonitis, carpal tunnel syndrome, or other musculoskeletal disorders.
- Only wear the device at night.

! DEVICE WARNINGS

- Do not expose the device to consumer products (e.g., perfume, insect repellent, etc.) containing ethanol that may damage the device.
- No modification of this equipment is allowed.
- The device can heat up during charging. Do not place on flammable surfaces when charging.
- Only use the Ava charging cable that was de-

- livered with the device to charge the device.
- Never use a damaged cable or charger to charge your device.

PLEASE SEE THE WARNINGS, PRECAUTIONS AND WARRANTY SECTION FOR ADDITIONAL IMPORTANT SAFETY AND PRODUCT INFORMATION!

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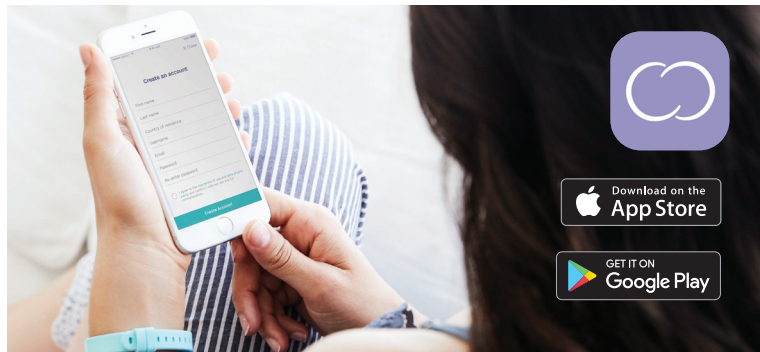
Download the Ava Fertility Tracker app from the App Store® or Google Play™ store and create an account.

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Pair the Ava Bracelet with the app by connecting it to a power source and following the onscreen instructions. The LED should come on when you plug the bracelet in.

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Charge the Ava Bracelet for at least four hours before its first use and for at least two hours daily, until the LED turns solid green. Only use the Ava charging cable provided to charge.

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Wear the bracelet on your non dominant wrist, below the bone, and always in the same strap notch. Tighten the band to a snug fit that doesn't slide. The device vibrates when ready.

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Start on any day of your cycle. Wear the Ava Bracelet at night for at least **four hours of sleep time**.

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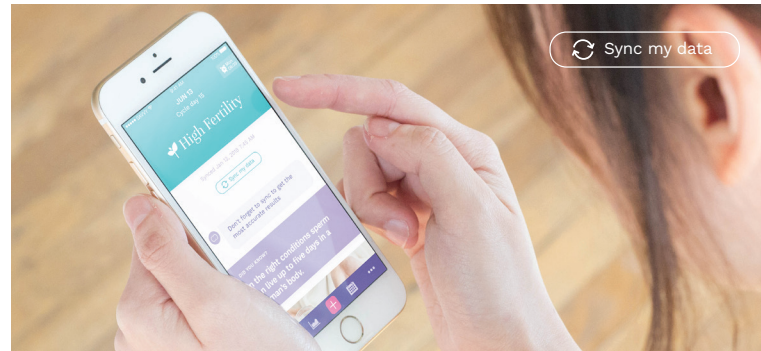
Retrieve your data every morning by opening the app and **tapping the “Sync” button** on the home screen.

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Consistently recording nightly data will give you the most accurate real-time information about your fertility.

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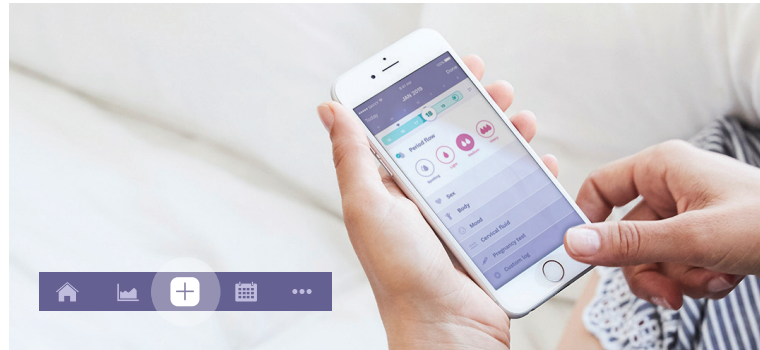
Log the start of each period in the app. (Don't skip this step as it marks the first day of your cycle!) Tap the “+” button, then select your flow.

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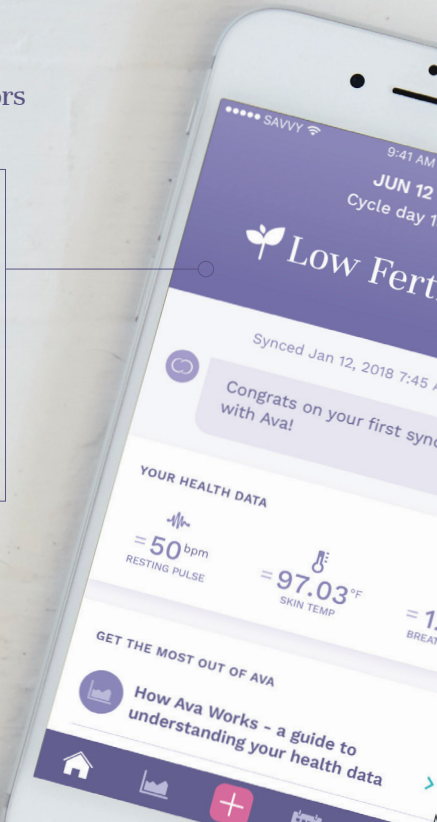
Ava Fertility Tracker

App homescreen color indicators

-  **Peak fertile day:**
highest chance of conceiving
-  **High fertile day:**
high chance of conceiving
-  **Low fertile day:**
low chance of conceiving
-  **Period / Low fertile day:**
low chance of conceiving

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Ava for Pregnancy

Once you get pregnant, keep using the app to learn how your body changes during pregnancy. In your profile select "I'm pregnant" and follow the onscreen instructions to transition to Ava for Pregnancy.

For more details download the Ava App Guide from www.avawomen.com/getstarted.

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The Ava Bracelet LED indicator

- Solid yellow = Bracelet needs to be synced
- Flashing yellow = Bracelet needs to be charged
- Flashing green = Charging in progress
- Solid green = Charging complete
- Solid green and vibration = Ava is ready to record data
- Flashing blue = Data is transmitting

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EN – IMPORTANT SAFETY & PRODUCT INFORMATION

Additional device warnings

- Do not wear your device while charging it.
- Do not charge a wet device.
- Only charge your device using chargers that are IEC 60950-1 or IEC 62368-1 compliant.
- This device cannot be serviced or repaired. Do not repair it yourself.
- This device consists of small parts that may be hazardous to children and pets. Keep the device out of reach.
- The Ava charging cable may be hazardous to children and pets. Keep the cable out of reach in order to prevent strangulation.

Device precautions

- Do not use the device if there is visible damage.
- The device contains a Li-Po battery. Do not attempt to remove the battery from the device.
- Do not dispose the device in a fire. The battery could explode.
- Do not apply mechanical stress.
- Do not open the enclosure or disassemble the device.
- Do not use the device more than 2 years after the first use.
- Do not interconnect this device with other equipment than equipment described in these instructions.
- The device is not waterproof (IP: 22). Avoid contact with any liquid.
- Avoid prolonged exposure of the device to direct sunlight.
- Use skin care products sparingly on the areas of your skin covered by the device.
- The device is not defibrillation-proof.

Operation precautions

- Always make sure to update your software app, so that you have the latest version. Ava recommends that you set your smartphone to update the app

automatically.

- Use WPA2 encryption or better on your Wi-Fi router when using Ava. Be sure to select a strong Wi-Fi password that can't be guessed by third parties.
- Only operate in environment with 15% - 90% relative humidity, non-condensing.
- Only operate in environment with ambient temperature between 5°C and 38°C (41°F and 100°F).
- Only operate in environment with atmospheric pressure between 70kPa and 106kPa (525mmHG and 795 mmHG).
- Do not use the device in oxygen rich environment.
- Remove the bracelet from your skin to stop recording physiological parameters.
- Lint, dust and light can influence the device's data recording.

Storage precautions

- Only store or transport the device in environment with 15% - 85% relative humidity, non-condensing.
- Only store or transport the device in environment with ambient temperature between -20°C and 35°C (-4°F and 95°F).
- Do not drop, knock, or shake the device. Rough handling and vibration can damage internal circuit boards and sensitive mechanical components.

Cleaning & disposal precautions

- Do not clean the device while wearing or charging.
- Make sure that the device is free from dirt and fluids.
- Make sure that the USB port of the device is free from dirt and fluids.
- Use a clean, damp cloth to clean the device occasionally from any visible debris. Never wash the device under a faucet.
- Never use harsh chemicals, powerful detergents, cleaning solvents (e.g. alcohol-based solvent) or ethanol.
- Dispose this device appropriately. Substances in this device and its battery may harm the environment or cause injury if handled and disposed improperly.

- Contact your local waste disposal department to dispose the device / batteries in accordance with applicable local laws and regulations.

ELECTROMAGNETIC EMISSIONS & IMMUNITY

This device is intended to be used in home healthcare environment (in bed at night) and not during transportation. The end user is the intended operator of this device.

The device is EMC-tested in conformity with the requirements of IEC 60601-1-2:2014 4th Edition.

Guidance and manufacturer's declaration electromagnetic emissions		
The Ava device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Ava device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ava device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test standard	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±15 kV air	±8 kV contact ±15 kV air	N/A
Electrical fast transient / burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines Not applicable (no input/output lines)	N/A
Surge IEC 61000-4-5	±1 kV line to line ±2 kV line to earth	±1 kV line to line ±2 kV line to earth	N/A
Voltage dips, short interruptions and voltage variations on power supply lines IEC 61000-4-11	<5% U _i (0,5 cycle) 40% U _i (5 cycles) 70% U _i (25 cycles) <5% U _i for 5 s	0% U _i (0,5 cycle) 0% U _i (1 cycle) 70% U _i (25/30 cycles) 0% U _i (250/300 cycles)	N/A
Note: U _i is the a.c. mains voltage prior to application of the test level.			
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	100 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ava device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such an environment.			
Immunity test standard	IEC 60601 test level	Compliance level	Recommended separation distance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150kHz to 80MHz outside ISM bandsa 6 Vrms 150 kHz to 80 MHz in ISM and amateur radio bands 10 V/m 80 MHz to 2.5 GHz	10 Vrms 150 kHz to 80 MHz 10 V/m 80 MHz to 2.7 GHz	Portable and mobile RF communications equipment should be used no closer to any part of the Ava device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter except as indicated in proximity field immunity information. $d = (3.5/V) \sqrt{P}$ 150 kHz to 80 MHz $d = (3.5/E) \sqrt{P}$ 80 MHz to 800 MHz $d = (7/E) \sqrt{P}$ 800 MHz bis 2.5 GHz Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range. b Interference may occur in the vicinity of equipment marked with the following symbol: 
Note 1: At 80 MHz and 800 MHz, the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Ava device is used exceeds the applicable RF compliance level above, the Ava device should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Ava device. b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between portable and mobile RF communications equipment and the Ava Device			
The Ava device is intended for use in the electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Ava device can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Ava device as recommended below, according to the maximum output power of the communication equipment except as indicated in proximity field immunity information.			
Rated maximum output power of transmitter (w)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz $d = 1.2 \sqrt{P}$	80 MHz to 800 MHz $d = 1.2 \sqrt{P}$	800 MHz to 2500 MHz $d = 1.2 \sqrt{P}$
0.01	0.12 m	0.12 m	0.23 m
0.1	0.38 m	0.38 m	0.73 m
1	1.2 m	1.2 m	2.3 m
10	3.8 m	3.8 m	7.3 m
100	12 m	12 m	23 m
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer. Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people. Note 3: An additional factor of 10/3 is used in calculating the recommended separation distance to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas.			

Radiated RF Immunity Proximity Fields		
Band (MHz)	IEC 60601 test level (V/m)	Compliance level (V/m)
385	27	27
450	28	28
710 745 780	9	9
810 870 930	28	28
1720 1845 1970	28	28
2450	28	28
5240 5500 5785	9	9

Radio Frequency Radiation Exposure

- This device is a mobile transmitter and receiver that uses an internal antenna to send and receive low levels of radio frequency (RF) energy for data communications.
- Bluetooth LE Communication.
- Frequency band: 2.400 to 2.4835GHz.
- The effective radiated power is below 1mW (0dBm).
- The operating range for effective transmission is 5 to 10 meters (16 to 32 feet).
- Europe: EU EN 300 328 V2.1.1
- USA: FCC 47 CFR Part 2 and Part 15.

Warnings

- Keep this product away from other high frequency devices which may interfere with operation.

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Ava Bracelet, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of the Ava Bracelet adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, the Ava Bracelet and the other equipment should be observed to verify that they are operating normally.

Precautions

- Avoid static electricity, ESD and high voltage as these may damage the device.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

Device information

- The rated input from the supply shall be 5 VDC/100 mA.
- The rated voltage range of the device is 4.75 – 5.25 VDC.
- The device does not need time to warm up or cool down from storage conditions before usage.
- Internally powered device; rechargeable Li-Po battery cell; 3.7 V nominal, capacity 80mAh.
- The Ava charging cable is 50cm (20 inches) long.

Ava LIMITED PRODUCT WARRANTY

This Limited Warranty Policy ("Warranty") is offered by Ava AG, a company formed under the laws of Switzerland with domicile in Zurich, Switzerland, and its subsidiary Ava Science, Inc., a Delaware corporation with principal business in San Francisco, California (collectively „Ava“ or „we“). The Warranty applies to the sale and use of Ava-branded fertility tracking bracelet marked with the „Ava“ trade name or logo and containing a valid Ava serial number which were sold/purchased on Ava's website ("Site") or from an authorized reseller ("Product").

This Warranty is also governed by the most current ver-

sion of the Terms of Service ("Terms"), which are located at www.avawomen.com/terms-of-service and are hereby incorporated into this Warranty by reference as though set forth in full herein. In case of a conflict between the Terms and the Warranty, the Terms shall prevail.

BY USING YOUR PRODUCT, YOU ARE AGREEING TO BE BOUND BY THE TERMS OF THIS WARRANTY. DO NOT USE YOUR PRODUCT UNTIL YOU HAVE READ THE TERMS OF THE WARRANTY. IF YOU DO NOT AGREE TO THE TERMS OF THE WARRANTY, DO NOT USE THE PRODUCT.

1. Scope of Warranty.

This Warranty applies only to the original purchaser ("you") who purchased the Product from an authorized source.

This Warranty gives you specific legal rights, and you may have other legal rights, including under consumer laws, that vary in different jurisdictions. To the extent permitted by law, this Warranty and the remedies set forth are exclusive and in lieu of all other warranties, remedies and conditions, whether oral, written, statutory, express or implied. Ava disclaims all statutory and implied warranties, including without limitation, warranties of merchantability and fitness for a specific purpose and warranties against hidden or latent defects, to the extent permitted by law. In so far as such warranties cannot be disclaimed, Ava limits the duration and remedies of such warranties to the duration of the Warranty (to the extent permitted by law) and, at Ava's option, the repair or replacement services described below. Some jurisdictions do not allow limitations on how long an implied warranty (or condition) may last, so the limitation described above may not apply to you.

2. Product Warranty.

Ava warrants that the Product will be free from material defects in workmanship and materials under normal use for a period of one (1) year from the date of your purchase, unless longer minimum periods are required under mandatory laws in your jurisdiction ("Warranty Period").

3. Remedies.

If you submit warranty claims to Ava before the end of the Warranty Period and your Product clearly and verifiably

fails the warranty, Ava will, in its sole discretion: (i) refund you in the amount of the price you paid for the Product (excluding taxes and shipping costs); (ii) exchange the Product with a new or refurbished Product (exchange the Product of identical specifications or functional equivalent); or (iii) repair the Product. This Section sets forth your sole and exclusive remedy, to the extent permitted by law. Returns shipped without Ava's prior express return authorization will not be accepted.

If a Product has been exchanged or repaired following this Warranty, it will be subject to a warranty for either: (i) the remaining original Warranty Period; or (ii) thirty (30) days from the date of repair or replacement or for any additional period that is applicable in your jurisdiction. This Warranty does not cover the software embedded in the Product and the services provided by Ava to owners of the Product, nor your use of the Site and the application provided by Ava. Please refer to the Terms for details of your rights with respect to their use.

4. Obtaining Warranty Service.

You must deliver the Product to the address specified by Ava in either its original packaging or packaging providing an equal degree of protection. In accordance with applicable law, Ava may require that you furnish proof of purchase details and/or comply with registration requirements before any warranty service is rendered. It is your responsibility to synchronize any data stored on the Product. It is likely that such data will be lost or reformatted during service, and Ava will not be responsible for any such damage or loss. Ava is not responsible for lost or damaged return shipments. You assume responsibility for the payment of any shipping charges and for all risk of loss or damage to the Product during shipment to Ava.

5. Exclusions and Limitations.

This Warranty does not apply to, and explicitly excludes, the following: (a) consumable parts, such as straps, unless damage has occurred due to a defect in materials or workmanship in the Product; (b) normal wear and tear and cosmetic damage, including, but not limited to, scratches, dents, damaged packaging material or damaged plastic parts; (c) defects or damage caused by alteration, unusual stress, accident (including, without limitation, collision, modification fire and the spillage of food or

liquid), neglect, abuse, misuse, unauthorized repair, installation, wiring, improper storage or testing, use of third party accessories such as straps or cables, third-party applications downloaded to the Product or if the serial number has been removed; (d) damage caused by using the Product outside the permitted or intended uses described by Ava; (e) damage caused by use with non-Ava products; (f) any non-Ava hardware products or any software, even if packaged or sold with the Product; and (g) modifications of the Product by anyone other than Ava. Ava reserves the right to disable Product if Ava suspects fraud or if Ava finds out that the Product is counterfeit, stolen or lost in shipment. This Warranty does not cover software embedded in the Product and the services provided by Ava to owners of the Product. Please refer to the Terms for details of your rights with respect to their use.

6. Implied Warranties.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE MANDATORY LAW, ALL IMPLIED WARRANTIES (INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A SPECIFIC PURPOSE) SHALL BE EXCLUDED OR, IF LEGALLY NOT PERMITTED, LIMITED IN DURATION TO THE DURATION OF THIS LIMITED WARRANTY.

Some jurisdictions do not allow limitations on the duration of an implied warranty, so the above limitation may not apply to you.

7. Limitation of Damages.

AVA SHALL NOT BE LIABLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL OR PUNITIVE DAMAGES WHETHER ARISING IN AN ACTION (INCLUDING WITHOUT LIMITATION LOSS OF REVENUE, USE, PROFITS, DATA, OR GOODWILL OR COSTS OF PROCURING SUBSTITUTE PRODUCTS) FOR BREACH OF CONTRACT, TORT, PRODUCT LIABILITY OR ANY OTHER CAUSE OR THEORY OF LIABILITY, RELATED TO OR IN CONNECTION WITH THIS WARRANTY, THE TERMS OR THE PURCHASE, SALE, USE, OPERATION OR PERFORMANCE OF THE PRODUCTS. AVA AND YOU HAVE AGREED THAT THESE LIMITATIONS WILL SURVIVE AND APPLY EVEN IF ANY LIMITED REMEDY SPECIFIED IN THESE TERMS IS FOUND TO HAVE FAILED OF ITS ESSENTIAL PURPOSE. YOU AGREE AND ACKNOWLEDGE THAT THE TOTAL AMOUNT OF AVA'S LIABILITY, IF ANY, FOR ANY

AND ALL CLAIMS, CAUSES OF ACTION, LOSSES, DAMAGES, OR JUDGMENTS ARISING OUT OF OR RELATED TO THIS AGREEMENT OR THE SERVICES SHALL NOT EXCEED THE COST OF THE PRODUCT AT ISSUE OR \$500.00, WHICHEVER IS GREATER.

Some jurisdictions do not allow the limitation or exclusion of liability for incidental or consequential damages, so the above limitation or exclusion may not apply to you.

©2020 AVA AG, all rights reserved. Ava and Ava logotype are trademarks of Ava AG. Bluetooth is a trademark of Bluetooth SIG Inc. iPhone and iOS are trademarks of Apple Inc., registered in the US and other countries. Android is a trademark of Google Inc. All other trademarks that are not mentioned by name are property of their respective owners.

Patent pending. Swiss design and technology.

Made in Europe.

For more information visit www.avawomen.com

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USA

FCC ID: 2APV7-AVA02

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference.
- (2) This device must accept any interference received, including interference that may cause undesired operation.

The Ava Fertility Tracker has been designed and complies with the safety requirements for portable RF exposure in accordance with FCC rule part §2.1093 and KDB 447498 D01.

Canada

IC: 23955-AVA02 Model 2.1

EN: This device complies with ICSED's license-exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable use conditions.

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Europe (EU/EFTA)

EN: Declaration of conformity with regard to the EU Directive 2014/53/EU.

Hereby, Ava AG declares that the radio equipment type PNA00046 is in compliance with Directive 2014/53/EU. The full text of the EU declaration of conformity is available at the following internet address: www.avawomen.com/quality

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SYMBOLS GLOSSARY (EN / DE / FR / ES / IT)

Symbols indicate all important instructions for safe use of this device. Ignoring these indications can lead to serious damage on the device or safety exposure for the user.

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SYMBOL	STANDARD REFERENCE	STANDARD TITLE	SYMBOL TITLE	EXPLANATORY TEXT
	ISO 15223-1, Clause 5.1.1	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Manufacturer	Indicates the medical device manufacturer.
	ISO 15223-1, Clause 5.1.3	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Date of manufacture	Indicates the date when the medical device was manufactured.
	ISO 15223-1, Clause 5.1.2	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Authorized representative in the European Community	Indicates the authorized representative in the European Community.
	ISO 15223-1, Clause 5.1.6	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Catalogue or model number	Indicates the manufacturer's serial number so that a specific medical device can be identified.
	ISO 15223-1, Clause 5.1.7	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Serial number	Indicates the manufacturer's catalogue number so that the medical device can be identified.
	ISO 15223-1, Clause 5.4.3	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Consult instructions for use	Indicates the need for the user to consult the instructions for use.
	IEC 60601-1, Table D.1, Symbol 11	Medical electrical equipment - Part 1: General requirements for basic safety and essential performance		

SYMBOL	STANDARD REFERENCE	STANDARD TITLE	SYMBOL TITLE	EXPLANATORY TEXT
	ISO 15223-1, Clause 5.4.4	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Caution	Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.
	IEC 60601-1, Table D.1, Symbol 10	Medical electrical equipment - Part 1: General requirements for basic safety and essential performance		
	ISO 15223-1, Clause 5.3.8	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Humidity limitation	Indicates the range of humidity to which the medical device can be safely exposed.
	ISO 15223-1, Clause 5.3.7	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Temperature limit	Indicates the temperature limits to which the medical device can be safely exposed.
	ISO 15223-1, Clause 5.3.9	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Atmospheric pressure limitation	Indicates the range of atmospheric pressure to which the medical device can be safely exposed.
	ISO 15223-1, Clause 5.3.4	Medical devices - Symbols to be used with medical device labels, labelling and information to be supplied - Part 1: General requirements	Keep dry	Indicates a medical device that needs to be protected from moisture.
	IEC 60601-1, Table D.1, Symbol 20	Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.	Type BF applied part	To identify a type BF applied part complying with IEC 60601-1.
IPN ₁ N ₂	IEC 60601-1, Table D.3, Symbol 2	Medical electrical equipment - Part 1: General requirements for basic safety and essential performance.	Degree of Ingress Protection Provided by Enclosure	Manufacturer-determined degree of particle and water ingress protection, where...
				N1 = 2: Protected against solid foreign objects of 12.5 mm (0.5 in) diameter and greater. N2 = 2: Protected against vertically falling water drops when enclosure tilted up to 15°.

SYMBOL	STANDARD REFERENCE	STANDARD TITLE	SYMBOL TITLE	EXPLANATORY TEXT
	EN 50419	Marking of Electrical and Electronic Equipment in accordance with Article 11(2) of Directive 2002/96/EC (WEEE).	Recycle: Electronic Equipment	DO NOT THROW IN TRASH.
	2017/745 (MDR)	Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices, amending Directive 2001/83/EC, Regulation (EC) No 178/2002 and Regulation (EC) No 1223/2009 and repealing Council Directives 90/385/EEC and 93/42/EEC	CE marking	Signifies European technical conformity.
	RED 2014/53/EU (Articles 19, 20, Annex II)	Directive 2014/53/EU of the European Parliament and of the Council of 16 April 2014 on the harmonisation of the laws of the Member States relating to the making available on the market of radio equipment and repealing Directive 1999/5/EC		
	ISO 7000 / IEC 60417 - reference number 6091	Graphical symbols for use on equipment	Keep out of reach of children	
	ISO 7000 - reference number 1135	Graphical symbols for use on equipment	Capable of being recycled	
	ISO 7000/IEC 60417 - reference number 5031	Graphical symbols for use on equipment	Direct current	To indicate on the rating plate that the equipment is suitable for direct current only; to identify relevant terminals.

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Ava AG
Gutstrasse 73, CH-8055
Zurich, Switzerland
www.avawomen.com

Representative in the USA
Ava Science, Inc.
77 Geary Street

Representative in the EU
Emergo Europe
Prinsessegracht 20
2514 AP The Hague, The Netherlands

Ava Bracelet version 2.1
Ava Armband Version 2.1
Bracelet Ava version 2.1
Pulsera Ava versión 2.1
Braccialetto Ava versione 2.1

User Manual PNA10039 / Version 2.6 Printed in China.
Benutzerhandbuch PNA10039 / Version 2.6 in China gedruckt.
Manuel de l'utilisateur PNA10039 / Version 2.6 Imprimé en Chine.
Manual del usuario PNA10039 / Versión 2.6 Impreso en China.
Istruzioni d'uso PNA10039 / Versione 2.6 Stampato in Cina.

© January 2021 Ava AG

For more information go to: www.avawomen.com
Weitere Informationen finden Sie unter: www.avawomen.com
Pour en savoir plus: www.avawomen.com
Si desea obtener más información, visita: www.avawomen.com
Per maggiori informazioni visitare: www.avawomen.com

Please contact the manufacturer in case of unexpected operation or event.

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Please contact the manufacturer, if needed, in setting up or using the device.

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For additional information about your device and the Ava Fertility Tracker app visit www.avawomen.com/getstarted

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